

DEPARTMENTAL MEMORANDUM



DEL MAR AVIONICS

TO: Don Woods

NO.: ENG-718-77

FROM: Jim Kaine

DATE: December 13, 1977

SUBJECT: PRELIMINARY OPERATING INSTRUCTIONS FOR
MODEL 1979P PRINTER/PLOTTER

cc: B. Thornton

1. Connect the cable from the Model 671 Pressurometer to the rear of the Model 1979P Printer/Plotter.
2. Depress the "POWER" switch on the front panel of the Model 1979P to turn it on.
3. Prepare the patient tape to be analyzed on the Model 660 Scanner.
4. Set the thumbwheel switches on the front panel of the Model 1979P to match the start time on the clock on Model 660.
5. Depress the "PRINT" switch on the front panel of Model 1979P. (This should cause the paper to advance one line.)
6. Start the patient tape on Model 660 in the X120 mode. (The green light on the "PRINT" switch will blink on and off each time a reading is accepted.)
7. When the tape is stopped or when 127 readings have been input, the accumulated data is automatically output in both tabular and graphic form.
8. To make additional copies of the output, set the thumbwheel switches to all zeroes and depress the "PRINT" switch. (Note that if the "PRINT" switch is depressed with the thumbwheel switches set to other than all zeroes, the internal storage is prepared to accept new data and the old data is lost.)
9. To analyze additional tapes, return to Step 3.

Jim Kaine
Jim Kaine

JK:rz