


THORACIC IMPEDANCE

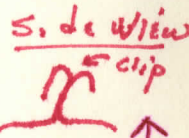

1. Unstow Sony, cassettes, measuring tape and electrodes
2. Remove shirt
3. Attach electrodes: *to: NECK: Chest - circumferential*
 Forehead - FH
 Neck - N *(circum)*
 Chest - C *(circum)*
 Abdomen - AB
4. Record subject, MET and vertical distance from chest to neck
5. Attach EKG electrodes to chest:
 Red Lead - left lower chest
 Green Lead - mid sternal chest
 White Lead - right lower chest
6. Don impedance belt, clip leads up
7. Attach impedance leads to corresponding electrodes FH, N, C, AB
8. Attach EKG leads to electrodes
9. Replace shirt
10. Connect red power cable to available DC UTIL outlet
11. DC UTIL PWR - ON
12. Attach orange output connectors to OBS
13. Attach output jack to Sony (red jack LINE IN/MIC)
14. Insert proper tape, label and record MET
15. Set recorder controls:
 REC LEVEL - max clockwise
 LINE IN/MIC - LINE IN
 TAPE - NORMAL
 DOLBY NR - OFF
 COUNTER - 000
 Simultaneously depress - PLAY, RECORD
16. IMPEDANCE BELT SWITCH - ON
17. Leave on for 5 min remaining quiet
18. If going to exercise:
 a. exercise for 3-5 min
 b. stop movement
 c. record for 15 sec
 d. continue exercise, stopping every 3-5 min to record
 e. at end of exercise record for 5 min
19. IMPEDANCE BELT SWITCH - OFF
20. SONY - STOP (white bar above play)
21. Record MET and footage on tape
22. DC UTIL PWR - OFF
23. Doff electrodes, stow equip

Measure chest tape & refer to belt

tell me again: circumferential lead ends overlap > Bend back a corner & clip with alligator clip?

no tape measure - red squares in locker - will label

will label (color not visible since new covering put on)

- AS ABOVE

need to see where located - will do in 1-6 at your convenience.

always reset tape see tape posn

need to know footage start & stop sheet for each subject.

Couldn't find tape electrodes

Why do I have any extra empty roller monitor tape? - shouldn't have 6 in there. There will be stowed an extra full roll of tape.

The cables are going to be a problem. I kind each sep by cover separately & put Velcro band around each individually. Electrode cables could be wound & velcroed to belt - will do

OBS - tell me what goes on OBS & what on Sony

Sony Bowel sounds Thoracic Impedance - Can belt go over abdominal strip? Yes

THORACIC IMPEDANCE DATA SHEET

SUBJECT				
MET				
CHEST TO NECK VERTICAL DISTANCE				
TAPE POSITION				

SUBJECT				
MET				
CHEST TO NECK VERTICAL DISTANCE				
TAPE POSITION				

SUBJECT				
MET				
CHEST TO NECK VERTICAL DISTANCE				
TAPE POSITION				

SUBJECT				
MET				
CHEST TO NECK VERTICAL DISTANCE				
TAPE POSITION				